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4490130_GA_1810

Notes for your replacement-pads for Smart-Massager

Art.Nr. 4490.1.30

CAUTION!

- Use the electrodes (pads) only for the smart massager No.art. 4490.1.00 or 4490.5.00.
- Please take note of the corresponding instructions concerning safety issues and use.

- Do not make any sharp kinks in the electrodes.
- After use, stick the electrodes onto the backing film.
- In order to ensure that the electrodes adhere as long as possible, they should be cleaned carefully with a damp, lint-free cloth.
- Electrode Pads are designed to be replaceable, so when the electrodes do not adhere firmly, you must replace electrode-pads.

Product Contents:

4 x Replacement-pads for product as mentioned above.

Cleaning:

- Please do not pull the wire.
- Please use clear water to wash or use wet cloth (instead of facial tissue) to gently wipe up the electrodes when cleaning them. Do not use brush or fingernail to clean surface of the electrodes to avoid scratching them.

Warranty Terms:

The following are excluded from the warranty: all defects caused by improper handling, damage or unauthorised attempts to repair. The same applies for normal wear.

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Hydas GmbH & Co. KG
c/o atrikom fulfillment GmbH • Haagweg 12